


The Breast Reduction Compression Device (BRCD) is a smart product to facilitate breast reduction surgeries. The product consists of an adjustable nylon strap encased in silicone tubing. The silicone tubing prevents the strap from slipping out of place while minimizing stress on the tissue.

All BRCD devices are delivered sterile in a double package.

All BRCD devices are CE marked and for single use only.

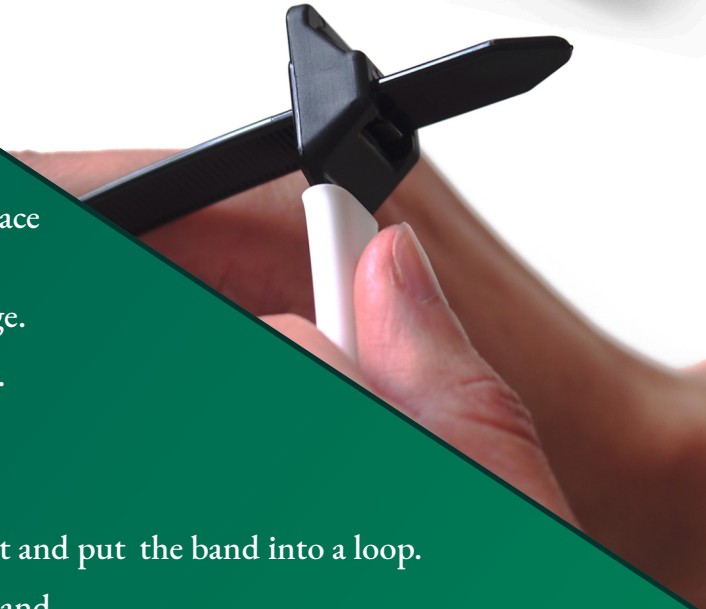

Procedure

- Wrap the BRCD around the base of the breast and put the band into a loop.
- Tighten the breast by contracting the band.
- Release the BRCD by pulling back the small plastic tab at the head.

Sizes

- | | |
|-------------------------|-------------------------|
| • BRCD Standard | • BRCD XL |
| 500-800 g reduction | Up to 1300 g reduction |
| Dimensions: 1.5 x 50 cm | Dimensions: 1.5 x 70 cm |

